

InView™ Silicone Male External Catheter

Introduction

This is a skills checklist for the application, and removal of the InView™ silicone male external catheter. The male external catheter (MEC) is intended to channel urine from the penis to extension tubing and urinary collection bags for adult males with urinary incontinence. The suitability of the MEC must be assessed for each patient. (See the section below entitled “Precautions” for factors to consider.)

Date Completed	Name	Employee Number	Dept.
----------------	------	-----------------	-------

Criteria (Refer to Instructions for Use for detailed information)	Eval Method*	Completed
Selecting the Proper Size and Preparation		
1. Use the color-coded Hollister sizing guide to determine correct size needed.		
2. Place the color-coded Hollister sizing guide beneath the widest diameter of the penis shaft (not the glans).		
3. In cases of shorter shaft length, select the InView Special catheter for better adhesion.		
4. When a larger adhesive band is needed to improve adhesion, select the InView Extra catheter.		
5. Trim pubic hair if necessary.		
6. Wash penis with mild soap and warm water. Dry thoroughly.		
Instructions for Use		
1. Open package at perforation. Remove sheath (MEC) from plastic insert, if present.		
2. Place rolled end over the end of the penis, leaving a small space between the end of the penis and the cone of the sheath (MEC). If applying to an uncircumcised penis, allow foreskin to remain in its natural position. Take care not to pinch or retract the foreskin.		
3. Slowly unroll onto the penis with as little wrinkling as possible until product is fully extended.		
4. Gently squeeze to properly seal adhesive to the skin. Do not attempt to reposition the product. If possible, avoid leaving a rolled ‘collar’ around the base of the penis. To connect this product to a drainage system, refer to the instructions for use packed with that product.		
To Remove and Dispose		
5. Disconnect from the drainage system. Gently roll the sheath (MEC) forward and off the penis. If necessary, apply a warm wet compress (such as a wet washcloth) around the sheath (MEC) to help loosen the adhesive. Please follow all local disposal regulations.		

* Evaluation Method: (L) return demos live in class or skill lab, (O) observation in worksetting, (CR) chart review, (V) verbalizes information, (R) read only – not performed – for information, (NA) not applicable.

Reviewed by (signature) _____

Warnings

- Reuse of a single use product may create potential risk to the user such as poor adhesion, leakage, and skin irritation.

Precautions

- Do not use on irritated or compromised skin.
- Do not use if allergic reaction occurs.
- Ensure proper fit as instructed. If the sheath (catheter) size is incorrect or leakage occurs, skin damage may occur (e.g., irritation, broken skin, rash, or redness). This may decrease wear time.
- Follow directions to remove if experiencing swelling, numbness, discomfort, pain, discoloration, or abnormal appearance.

Notes

- For good hygiene, change sheath (catheter) daily.
- To help ensure a proper adhesion, apply to clean, dry intact skin. Do not use lotions, powders, or ointments on the skin as these products may interfere with adhesion of the product.
- Contact a healthcare practitioner for help with the product or with skin irritation.

Prior to use, be sure to read the Instructions for Use for information regarding Intended Use, Contraindications, Warnings, Precautions and Instructions.

The Hollister logo and InView are trademarks of Hollister Incorporated.

© 2021 Hollister Incorporated. US-01431

Hollister Incorporated
2000 Hollister Drive
Libertyville, Illinois 60048
1.800.323.4060

www.hollister.com

